


Generic Evacuation Chair Quarterly Inspection Check List

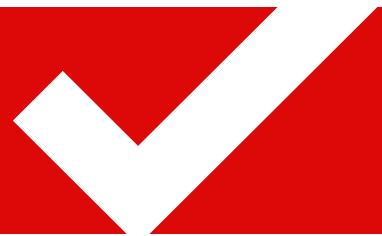

Is the instruction manual present?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Is the evacuation chair fixed securely to the wall?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<i>If your evacuation chair has a dust cover</i> Is it in place and free from cuts, tears or other damage?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Does the evacuation chair unfold smoothly and easily?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Is the seat material clean and free from cuts, tears or other damage?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Is the seat material secure at the fixing points?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Are the locking clips in good condition? Do they open and close freely?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Is the seatbelt free from damage and does it operate correctly ? <i>Check any connectors, fastenings and the release button</i>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Do the rear wheels run freely?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<i>If your evacuation chair has rear wheel breaks</i> Do they unlock and lock easily and freely?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Are all nuts and bolts secure and in place?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Do all exposed nuts and bolts have a protective cap or cover?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tracked Evacuation Chairs Do the track belts move smoothly around the runners?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tracked Evacuation Chairs Do the track belts rotate around the wheels?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tracked Evacuation Chairs Are the top handle locking clips secure and do they open freely?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tracked Evacuation Chairs Do the top handle locking clips lock the handle into each position?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Tracked Evacuation Chairs Does the track move smoothly around the runners and do they rotate around the wheels?	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Date of inspection	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>
Inspectors Initials	<input type="text"/>	<input type="text"/>	<input type="text"/>	<input type="text"/>

